

# Volumed<sup>®</sup> VP7000 PVC Premium Infusion Pump

# Trouble Shooter



Remove air bubble in; restart pump.



Occlusion Downstream: Check the following:  
 Is the door shut?  
 Is the line / cannula kinked?  
 Has the cannula tissued  
 Has the patient moved to stop flow?  
 Is the recommended IV set installed?  
 If no pressure in system, open and close door  
 If flashing; restart pump



Occlusion Upstream  
 Exchange container if empty  
 Is the drip chamber correctly installed?  
 Is the drip chamber vertically inserted in drop detector?  
 Is the drip chamber over filled? (Over 1/3 filled)  
 Are the walls of drip chamber clean? (No condensed drops)  
 Is the drop detector above fluid level and below white collar?

**DEFECT**

Do not use the pump any more  
 Write down exact circumstances  
 Send pump to Service Department

**RATE**

Rate has been changed

**KVO**

(Keep Vein Open) Rate = 3ml/h  
 Restart pump if desired



(GREEN Battery): Pump running on battery  
 (FLASHING): Pre Low Battery Alarm (30mins left)  
 Press **MENU/SELECT** key (18) to confirm




(RED Battery FLASHING & Audible Alarm):  
 Connect mains power as soon as possible



Check alarms  
 Confirm with Menu/Select Key

## Alarm Symbols

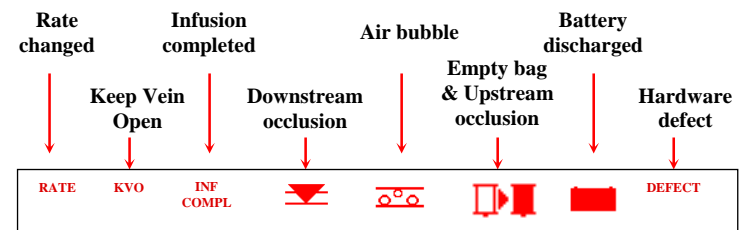

## Administration sets available for gravity or VP7000 Premium infusion pump

- 3100 series with Robson Clamp



### Arcomedical infusion Ltd

Head Office – Sales & Services  
 5J West Horndon Industrial Estate  
 West Horndon Essex CM13 3XL

Admin Email: [sales@arcomed.co.uk](mailto:sales@arcomed.co.uk) Website: [www.arcomed.co.uk](http://www.arcomed.co.uk)

Tel: 01277 810432

Fax: 01277 811967