



Syringe Preparation

- ✓ Prepare syringe

Loading syringe into the pump

- ✓ Press blue catch down (1) to open door
- ✓ Pull back syringe clamp (2)
- ✓ Wait until actuator (drive head) movement ceases (3)
- ✓ Insert syringe (4) (Ensure ears of syringe fit and make contact with front face of recess)
- ✓ Push syringe clamp (2) onto barrel of syringe and close door
- ✓ Press & hold **PRIME/BOLUS** key (8) drive head automatically engages onto syringe plunger.
- ✓ Press **START/STOP** key to confirm size & make of syringe, shown in the display (6)

Priming the line - starting and stopping Infusion

- ✓ **Prior to attaching extension line to cannula, attach line to syringe**
- ✓ Press and hold **PRIME** key to prime line (8) (Optional)
Note: Amount of delivery is dependent on how long the prime key is held
- ★ (see *Trouble Shooter* for rate calculation)
- ✓ Set required rate (Use keys in **RATE ML/H** window) (9)
- ✓ Set required volume (**VBTI**) (Use keys in **VOLUME ML** window) (10) (Optional)
- ✓ Press **START / STOP** key to begin infusion (7)
- ✓ To stop infusion at any time. Press **START/STOP** key (7) pump reverts to **KVO** rate 0.3 ml/hr

Removal of the syringe from the pump

- ✓ Press **START/STOP** key (1) to stop pump if pump is infusing
- ✓ Press blue catch down (1) to open door
- ✓ Pull back syringe clamp (2)
- ✓ Wait for actuator (drive head) to release the syringe and open completely (3)
- ✓ Remove syringe (4)
- ✓ Push syringe clamp back (2) and close door
- ✓ Press **PRIME/BOLUS** key (8) drive head will automatically park itself in the closed position
- ✓ Press and hold **ON / OFF** key for 2 seconds to power down syringe pump (11)

Sales and Service

Tel: 01277 810432
Fax: 01277 811967

Arcomedical Infusion Ltd.