



Loading syringe into the pump

- Press blue catch down (1) to open door
- Pull back syringe clamp (2) Wait until actuator (drive head) (3) movement ceases
- Insert syringe (4) (Ensure ears of syringe fit and make contact with front face of recess)
- Push syringe clamp (2) onto barrel of syringe and close door
- Press & hold **PRIME/BOLUS** key (8) drive head automatically engages onto syringe plunger.
- Press **START/STOP** key to confirm size & make of syringe, shown in the LCD display (6)

Priming the line

Prior to attaching extension line to cannula, attach line to syringe

- Press and hold **PRIME** key (8) to prime line
- Note: Amount of delivery is dependent on how long the prime key is held**

Press **START/STOP** key.

Starting PCA Infusion (Pre-Programmed Parameters)

- Enter 4 digit password – Confirm by pressing **START/STOP** key
- Select desired patient programme in **VOLUME ML** window (10)
 - 0 – Individual programme
 - 1 – 8 Pre-selectable programme
- Patient programmes will be displayed on the lower line of the LCD display (6)

After selecting the desired patient programme, the values of all parameters Pt.2 – Pt.15 in the **RATE ML** window (9) **must** be confirmed. Chosen parameter values will be shown in the **VOLUME ML** (10) and LCD window (6)

After Pt. 15, confirm the patient programme is correct by pressing **START/STOP** key. Patient programme will be confirmed on the upper line of the LCD display (6)

Start programme by pressing **START/STOP** key

Pump will now be running in selected programme.