


## 1. Priming – volumed IV Set


- ✓ Connect volumed IV set to fluid container
- ✓ Fill drop chamber (1/3 full) **(B)**
- ✓ Prime line (To remove air)
- ✓ Close roller clamp **(C)**
- ✓ Open air vent cap if using rigid container **(A)**

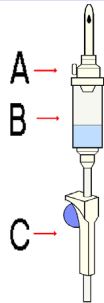

**Note: Use only volumed 8100 series Set**

## 2. Loading – volumed IV set in pump


- ✓ Clamp flow sensor over drip chamber **(A)**  
(Above fluid level & below white collar)
- ✓ Open door **(B)** & push back red lever **(C)**
- ✓ Install tubing into pumping chamber  
(Flow corresponds to the arrow direction)

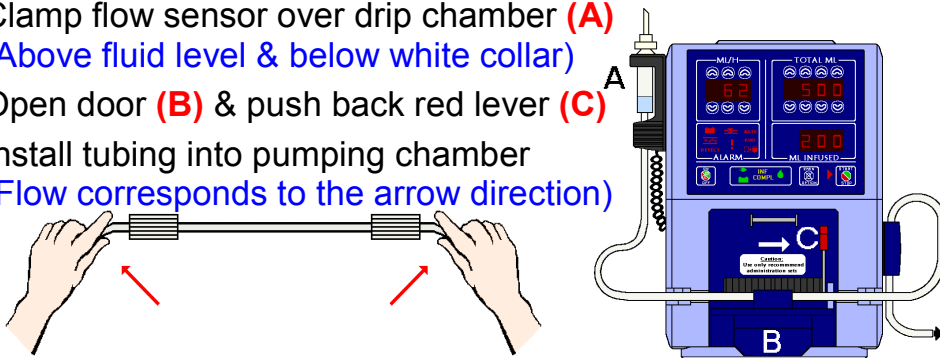

**Note: Keep IV straight between tubing guides**

- ✓ Close Door. Open roller clamp.


## 3. Loading – volumed IV set in Air in Line Detector

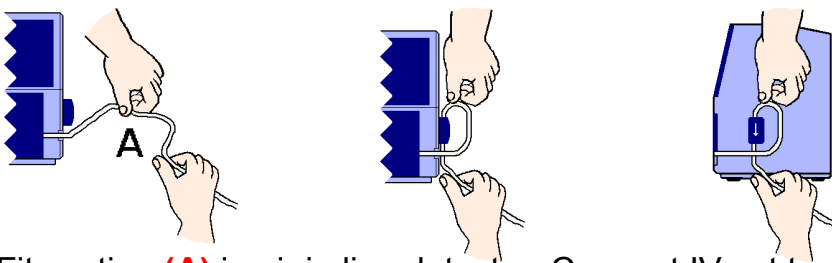

- ✓ Fit section **(A)** in air in line detector. Connect IV set to patient.

## 4. Starting an Infusion

- ✓ Press **ON/OFF** key (Switches power on) **(6)**
- ✓ Set required rate  
(Use keys in **ML/H** window) **(1)**
- ✓ Set required volume  
(Use keys in **TOTAL ML/H** window) (Optional) **(2)**
- ✓ Release roller clamp
- ✓ Press **START / STOP** key **(4)**  
(Dots run across the **ML INFUSED** window) **(3)**  
(Confirms pump is running)
- ✓ To stop pump press **START/STOP** key **(4)**
- ✓ To switch pump off, first stop pump, then press **ON/OFF** key **(6)**

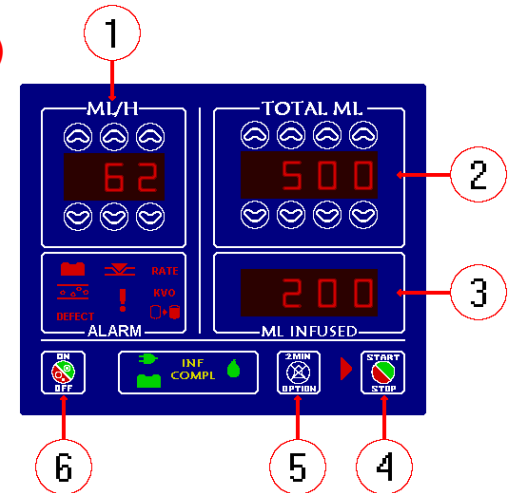

## 5. Additional Functions

- **Silence Alarm:** Press **2MIN** key **(5)**
- **Zero ML INFUSED Display:** Stop pump (**START / STOP** key) **(4)**  
Press **2MIN** key **(5)** & hold (Until R.SET is displayed)  
Immediately press **START / STOP** key **(4)** (To zero display)
- **Memory Recall:** (Last settings) (Pump in switched off state)  
Press & hold **START / STOP** key **(4)**, then press **ON / OFF** key **(6)**  
(Pump scrolls through menu and display last settings)
- **Rate Calculation:** Press & hold **2MIN** key **(5)**  
Press **ON / OFF** key **(6)**  
Enter VTBI, then rate  
(Time displayed in **ML / INFUSED** window) **(3)**
- **Pump in 0.1 mL Increment:** Press and hold **1mL** (Down key)  
Press **ON / OFF** key (ML / H displayed in 0.0)