





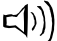
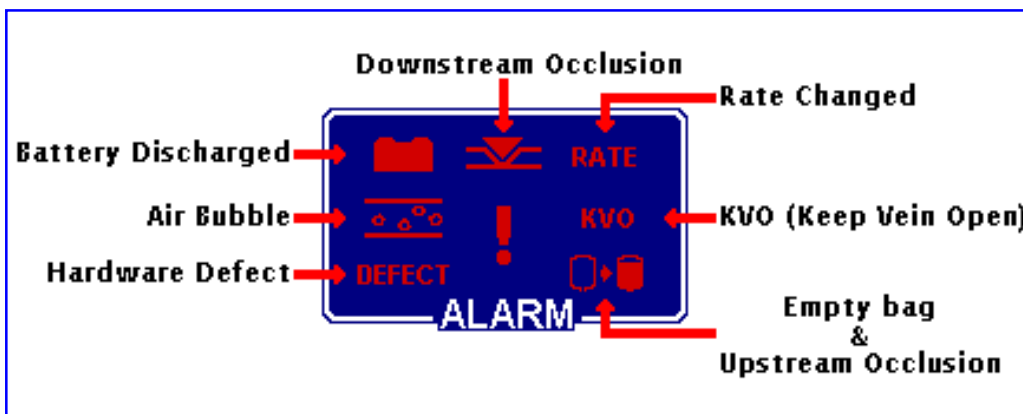


-  Remove air bubble in tubing in air detector  
Insert tubing firmly in air detector  
If flashing: restart pump
-  Occlusion Downstream: Check the following:  
Is the door shut?  
Is the line / cannula kinked?  
Has the cannula tissued  
Has the patient moved to stop flow?  
Is the recommended IV set installed?  
If no pressure in system, open and close door  
If flashing: restart pump
-  Occlusion Upstream  
Exchange container if empty  
Is the drip chamber correctly installed?  
Is the drip chamber vertically inserted in drop detector?  
Is the drip chamber over filled? (Over 1/3 filled)  
Are the walls of drip chamber clean? (No condensed drops)  
Is the drop detector above fluid level and below white collar?  
Is the roller clamp released?
- DEFECT** Do not use the pump any more  
Write down exact circumstances  
Send pump to Service Department
- RATE** Rate has been changed
- KVO** (Keep Vein Open) Rate = 3mL/h or 0.3ml/h in micro mode  
Restart pump if desired
-  Check if other alarms are displayed  
Check if air and pressure  
If interfaced with computer: pump stopped by computer
-  (GREEN Battery): Pump running on battery  
(FLASHING): Pre Low Battery Alarm (30mins left)  
Press **2MIN / OPTION** key (**5**) to confirm
-  (RED Battery FLASHING & Audible Alarm):  
Connect mains power as soon as possible
-  Check alarms  
Confirm with **2MIN** Key

## Alarm Symbols



## Administration Sets available for gravity or VP5005 infusion

- 8101P      **Solution set & lower Y site**
- 8101PK    **Solution set & upper Y site**
- 8101T      **Transfusion set & lower Y site (200 µ filter)**
- 8110        **Burette set (150ml chamber)**
- 8101E      **Epidural set**
- 8101A      **Solution set & air bubble & particle filter**

## Hospital, Ward etc Tel.

**arcomedical infusion ltd**  
Head Office – Sales & Services  
5j West Horndon Industrial Estate  
West Horndon, Essex CM13 3XL  
Email: [sales@arcomed.co.uk](mailto:sales@arcomed.co.uk) Website: [www.arcomed.co.uk](http://www.arcomed.co.uk)  
Tel: 01277 810432 Fax: 01277 811967