




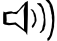
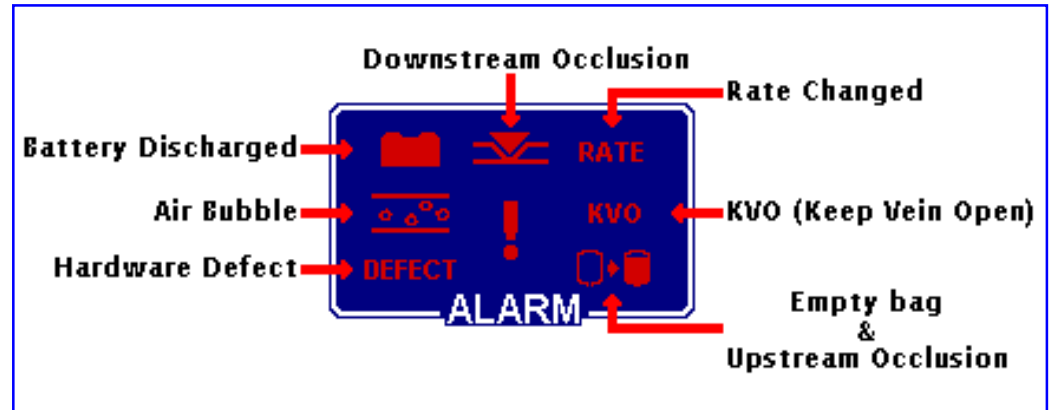
	Remove air bubble in tubing found behind air detector clip Ensure tubing is firmly inserted in air detector clip If flashing: restart pump
	Occlusion Downstream: Check the following: Is the door shut? Is the line / cannula kinked? Has the cannula tissued Has the patient moved to stop flow? Is the recommended IV set installed? If no pressure in system, open and close door If flashing: restart pump
	Occlusion Upstream Exchange container if empty Is the drip chamber correctly installed? Is the drip chamber vertically inserted in drop detector? Is the drip chamber over filled? (Over 1/3 filled) Are the walls of drip chamber clean? (No condensed drops) Is the drop detector above fluid level and below white collar? Is the roller clamp released?
DEFECT	Do not use the pump any more Write down exact circumstances Send pump to Service Department
RATE	Rate has been changed
KVO	(Keep Vein Open) Rate = 3ml/h or 0.3ml/h in micro mode Restart pump if desired
	Check if other alarms are displayed Check for air in line and occlusion pressure If interfaced with computer: pump stopped by computer
	(GREEN Battery): Pump running on battery (FLASHING): Pre Low Battery Alarm (30mins left) Press 2MIN / OPTION key (5) to confirm
	(RED Battery FLASHING & Audible Alarm): Connect mains power as soon as possible
	Check alarms Confirm with 2MIN Key

Alarm Symbols



Administration Sets available for gravity or VP5005 infusion

- **3101-P** (Blue Pack) Solution set & lower Y site
- **3101-T** (Red Pack) Transfusion set (200 µ filter)
- **3110** Burette set (150ml chamber)
- **3101-TOP** Combination primary secondary solution set & check valve
- **3101-A** Air Elimination & particulate filter set
- **3101-E** Epidural set
- **N.B.** A wider range on request

Hospital, Ward etc Tel.

arcomedical infusion ltd

Head Office – Sales & Services
5j West Horndon Industrial Estate
West Horndon, Essex CM13 3XL

Admin Email: sales@arcomed.co.uk Website: www.arcomed.co.uk

Tel: 01277 810432 Fax: 01277 811967