


	Remove air bubble in; restart pump.
	<p>Occlusion Downstream: Check the following:</p> <ul style="list-style-type: none"> Is the door shut? Is the line / cannula kinked? Has the cannula tissued Has the patient moved to stop flow? Is the recommended IV set installed? If no pressure in system, open and close door If flashing; restart pump
	<p>Occlusion Upstream</p> <ul style="list-style-type: none"> Exchange container if empty Is the drip chamber correctly installed? Is the drip chamber vertically inserted in drop detector? Is the drip chamber over filled? (Over 1/3 filled) Are the walls of drip chamber clean? (No condensed drops) Is the drop detector above fluid level and below white collar?
DEFECT	<p>Do not use the pump any more</p> <ul style="list-style-type: none"> Write down exact circumstances Send pump to Service Department
RATE	Rate has been changed
KVO	<p>(Keep Vein Open) Rate = 3.0 ml/h</p> <p>Restart pump if desired</p>
	<p>(GREEN Battery): Pump running on battery</p> <p>(FLASHING): Pre Low Battery Alarm (30mins left)</p> <p>Press 2MIN / OPTION key (18) to confirm</p>
	<p>(RED Battery FLASHING & Audible Alarm):</p> <p>Connect mains power as soon as possible</p>
	<p>Check alarms</p> <p>Confirm with 2MIN Key</p>

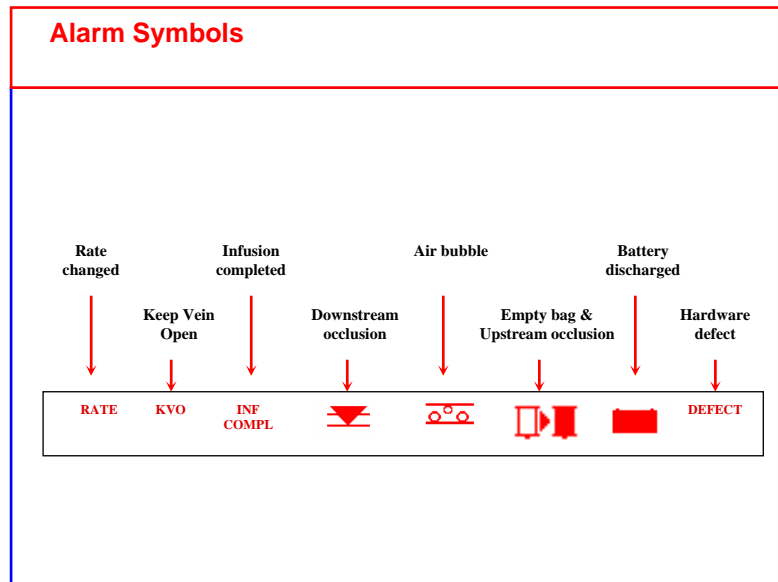

Administration sets available for gravity or VP7000

- **8100 series with Robson Clamp**

Arcomedical infusion ltd
 Head Office – Sales & Services
 5j West Horndon Industrial Estate
 West Horndon Essex CM13 3XL
 Admin Email: sales@arcomed.co.uk Website: www.arcomed.co.uk
 Tel: 01277 810432 Fax: 01277 811967 std.a5.5005.is